


Instruction on how to make a Cloth Face Protector

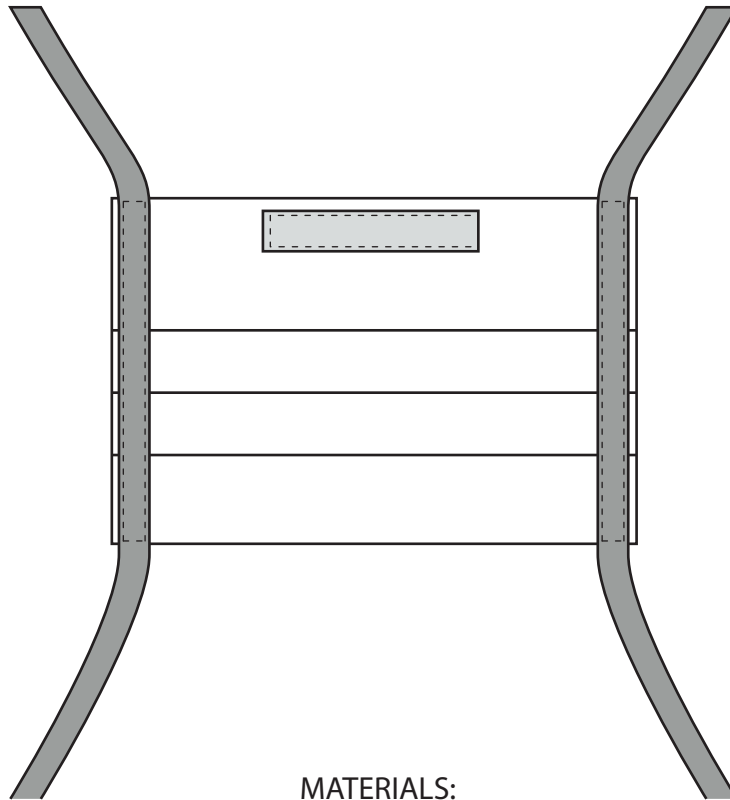

MATERIALS:

MAIN FACE PROTECTOR:

9" x 14" - 100% Pre-Washed Woven Cotton Fabric

-Fabric MUST have a pattern to it.

- NO blue & white patterns, NO solid blue, NO solid white, NO solid colors.

This is to avoid confusion with medical grade masks.

NOSE PIECE:

4" x 3/4" Twill Tape

- or -

4" x 1.25" Fabric

TIES:

48" Long (x2) - 1/2" Soft Grosgrain Ribbon

- or alternately -

1/2" or 1/4" Cotton Twill Tape or bias tape


- or -

binding made from 1" wide strips of fabric - turned into binding.


MATERIALS:

- Main Face Protector: 9" x 14" - 100% Pre-Washed Woven Cotton Fabric
 - Fabric MUST have a pattern to it.
 - NO blue & white patterns, NO solid Blue, NO Solid White, NO solid colors.
 - **This is to avoid confusion with medical grade masks.**
- Nose Piece: 4" x 3/4" Twill Tape - or - 4" x 1.25" Fabric
- Ties: 48" Long (x2) - 1/2" Soft Grosgrain Ribbon (or alternately 1/2" or 1/4" Cotton Twill Tape/Bias Tape
 - or - binding made from 1" wide strips of fabric)


STEP 1:
Cut fabric to 9" x 14"

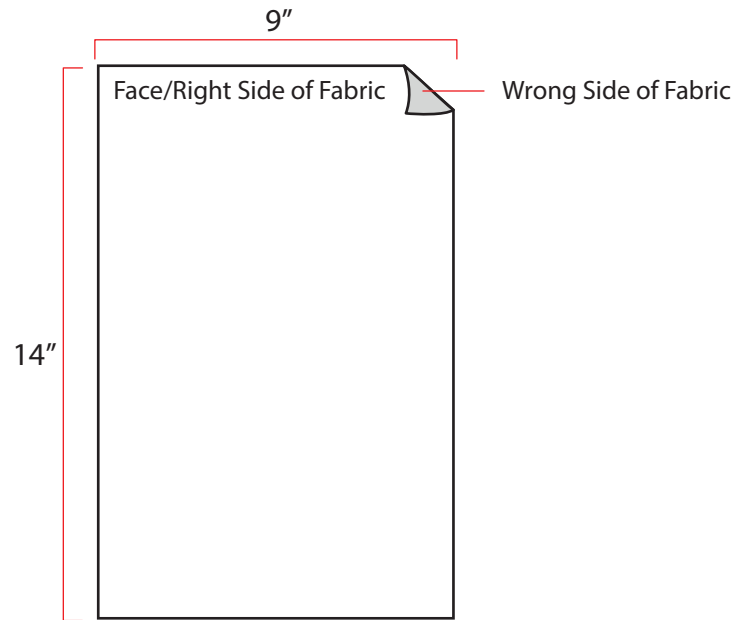

STEP 2:
Fold fabric in half.
Right sides together.

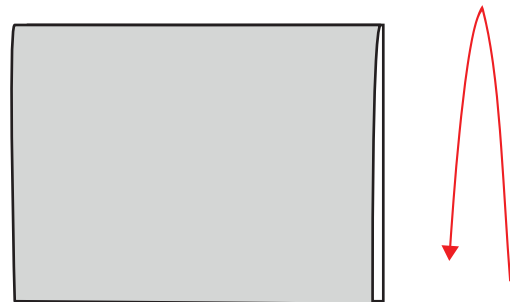

STEP 3:
Sew around open edges with 1/4" seam allowance.
Leave 1-2" open on side
to turn right side out.

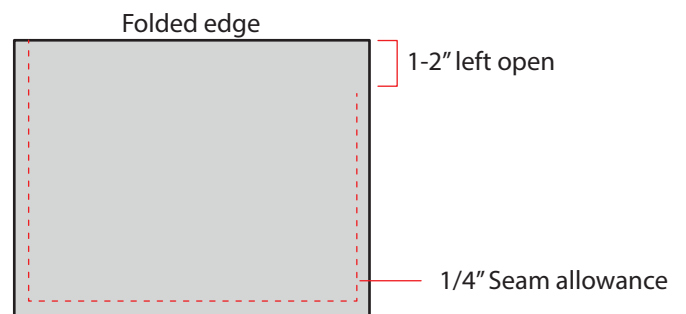


STEP 4:
Turn right side out - and sew closed hole.
Press flat.



Edge Stitch Opening Closed

STEP 5:
Prep for pleats.


From the bottom place pins (red pins):
2"
3.5"
5"

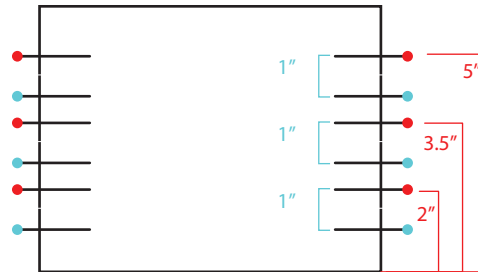


Next, place pins (blue pins) 1" down from each of the red pins.

Repeat on other side.

STEP 6:
Create pleats by folding the red pin on top of the blue pin.
Once folded - each pin should be about 1/2" apart.

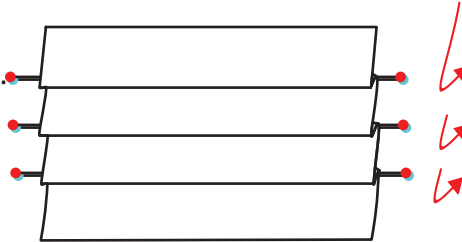

STEP 7:
Sew Pleats
Once sewn mask should measure about 3.5"x8.5"


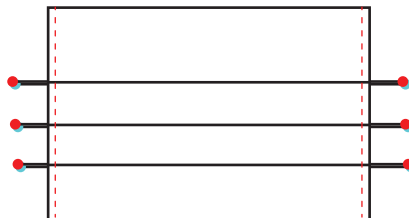


STEP 7 : TIES - 1/2" grosgrain ribbon

Apply ties on **FRONT** of mask.

(or alternately, 1/4" or 1/2" twill tape)

Or to make your own ribbon out of fabric:

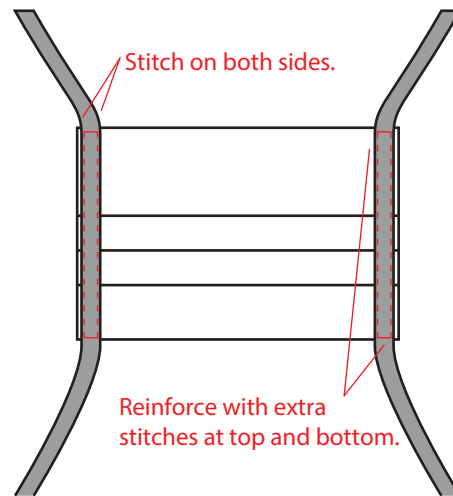
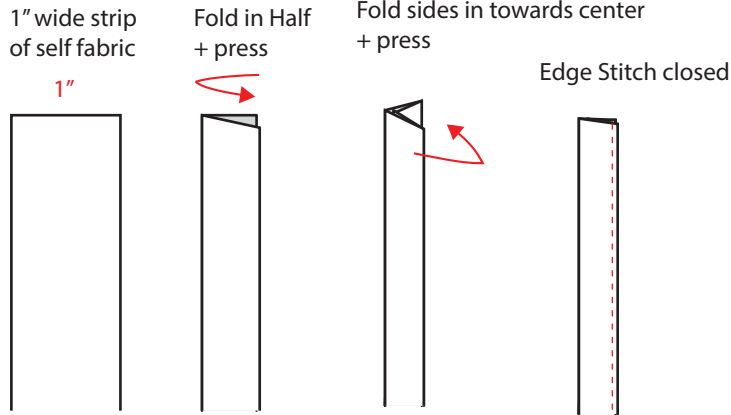


STEP 7: ALTERNATE TIE OPTION - 1/2" Twill tape.

Find center point of tie, and center point of side of mask.

Fold twill tape around side of mask. Sew. Be sure to reinforce at ends.

If using self fabric for binding + ties:

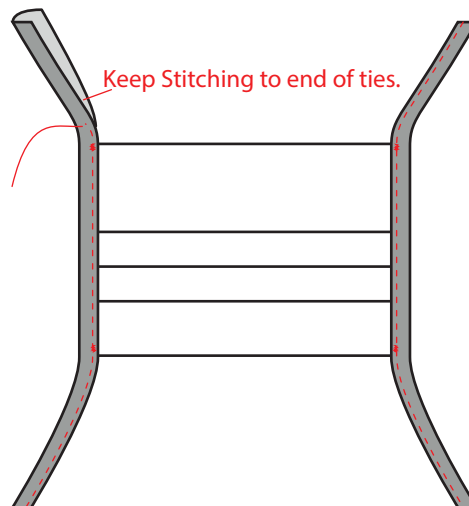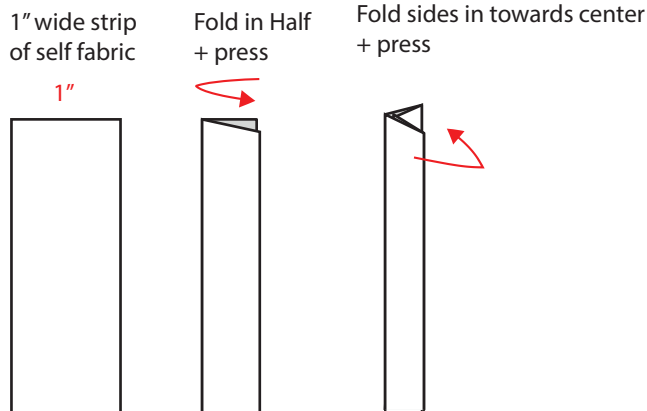

STEP 8:
Nose strip casing

You can use 3/4" Wide twill tape.
-or -
Cut 4" X 1.25" piece of fabric.
Turn sides under 1/4" and press.

Center nose strip casing on long side of mask
and position 1/4" down from top edge.

Edge stitch around nose strip on three sides -
leaving one short side open.

